INSTRUCTIONS FOR USE

INTERMEDIARY TUBE

WITH CHECK VALVE

for AQUA VISION[®]

REF. AV1102


Hemodia SAS
85 Rue du Chêne Vert
31670 Labège - FRANCE
hemodia@hemodia.com


103786_B
Date de révision : 09 DEC 2020

INTERMEDIARY TUBE WITH CHECK VALVE
for AQUA VISION®

REF. AV1102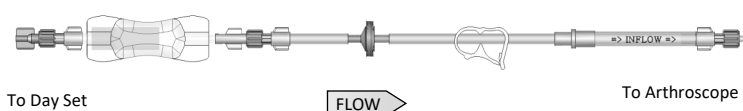

EN	INTERMEDIARY TUBE WITH CHECK VALVE for AQUA VISION®	4
ES	TUBO INTERMEDIO CON VÁLVULA ANTIRRETORNO para AQUA VISION®	5
IT	TUBO INTERMEDIO CON VALVOLA UNIDIREZIONALE per AQUA VISION®	6
DE	ZWISCHENGESCHALTETER SPÜLSCHLAUCH MIT RÜCKSCHLAGVENTIL für AQUA VISION®	7
FR	TUBE INTERMÉDIAIRE AVEC VALVE ANTI-RETOUR pour AQUA VISION®	8
NL	MIDDELSTE SLANG MET EEN-WEGKLEP voor AQUA VISION®	9
PT	TUBO INTERMEDIO COM VÁLVULA UNIDIRECCIONAL para AQUA VISION®	10
DA	MELLEMSLANG MED ÉNVEJSVENTIL til AQUA VISION®	11
NO	MELLOMLEDDSSLANG MED TILBAKESLAGSVENTIL for AQUA VISION®	12
FI	INTERMEDIAARINEN PUTKI SULKUVENTTIILIN KANSSA AQUA VISION®	13
EL	ΣΩΛΗΝΑΣ ΜΕ ΒΑΛΒΙΔΑ ΑΝΤΕΠΙΣΤΡΟΦΗΣ για AQUA VISION®	14
SV	MELLANSLANG MED BACKVENTIL till AQUA VISION®	15
PL	RURKA PŁUCZĄCA POŚREDNIA Z ZAWOREM ZWROTNYM DO POMPY AQUA VISION®	16

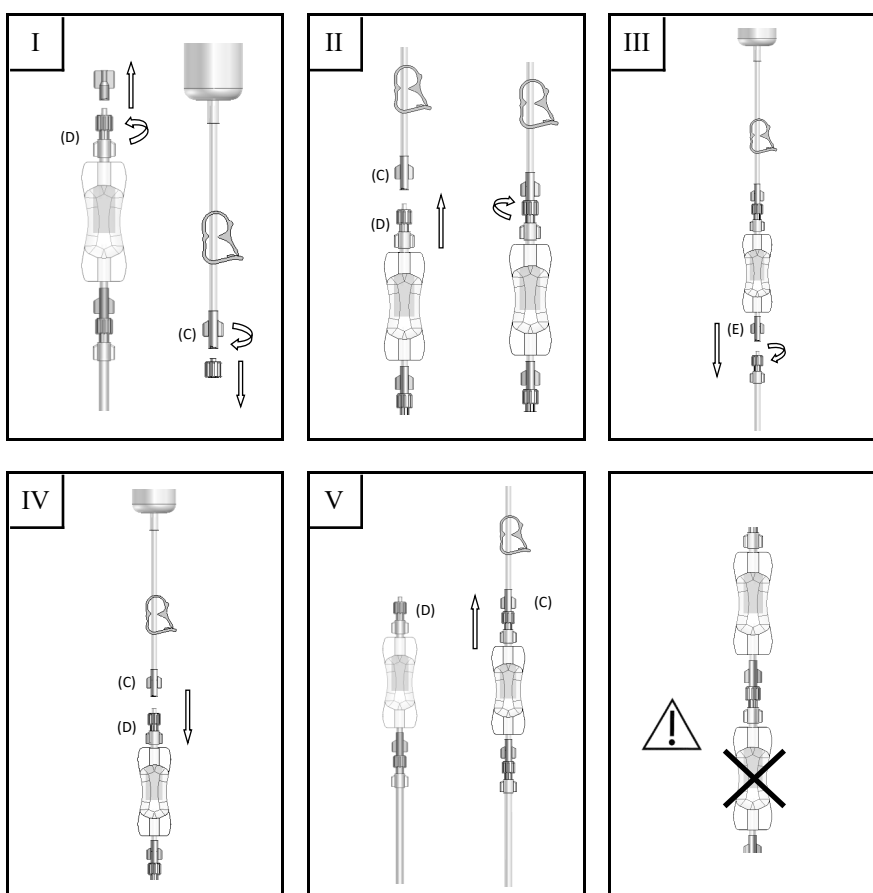INTERMEDIARY TUBE WITH CHECK VALVE
for AQUA VISION®
REF. AV1102

ENGLISH

INSTRUCTIONS FOR USE

INDICATIONS FOR USE: The device is intended to be used in conjunction with the Hemodia arthroscopic AQUA VISION® pump and delivers the irrigation fluid from the DAY SET tube (AV4503) to the arthroscope during arthroscopic procedures.

DESCRIPTION: INTERMEDIARY tube set for AQUA VISION® pump.

SUPPLY DETAILS: Sterile, single use. Sterile if the package is unopened and undamaged.

INSTRUCTIONS FOR USE:

1. Open using aseptic technique.
2. Open the outer package.
3. The scrub nurse removes the tube set from the package.
4. Ensure that all Luer locks are tight.
5. The scrub nurse hands the tubing end with the blue cap to the circulating nurse.
6. Remove the blue cap from the end of the INTERMEDIARY tubing (Fig I. D) and the DAY SET tubing (Fig I. C).
7. Immediately connect the INTERMEDIARY tubing (Fig II. D) to the DAY SET tubing (Fig II. C). Tighten the nut to lock the both Luer connectors.
8. Press on the "Run/Stop" button of the AQUA VISION® pump. Blinking green light will turn to solid green light. Open the clamp under the fill chamber to prime tubing.
9. Once the tubing has been primed, close off the clamp.

END of the operation:

Close the clamp under the pressure chamber

Disconnect the white Luer-lock (Fig III. E) under the check valve and discard tubing. The check valve will remain connected to the DAY SET tubing until the start of the next operation.

START of the next operation:

1. The circulating nurse removes the old check valve by disconnecting the used check valve (FIG IV. D) from the DAY SET tubing (FIG IV. C).
2. Immediately connect the blue Luer-lock (Fig. V. D) of the new INTERMEDIARY tubing to the blue connector (FIG V. C) of the DAY SET tubing as per step 7 in the Instructions for use.

WARNING:

1. Do not use if the package is damaged or opened.
2. Refer to and read the AQUA VISION® pump instruction manual for proper use.
3. This product is for single-use only. It has not been designed to be re-used/re-sterilized. Reprocessing may lead to changes in material characteristics such as deformation and material degradation which may impact the strength of the device and compromise device performance. Reprocessing of single use devices can also cause cross-contamination leading to patient infection. These risks may potentially affect patient safety.
4. Never attach one check valve to another.

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