INSTRUCTIONS FOR USE


DAY SET
for AQUA VISION®
REF. AV4503

Hemodia SAS
85 Rue du Chêne Vert
31670 Labège - FRANCE
hemodia@hemodia.com

103787_D
Date de révision : 28 SEP 2023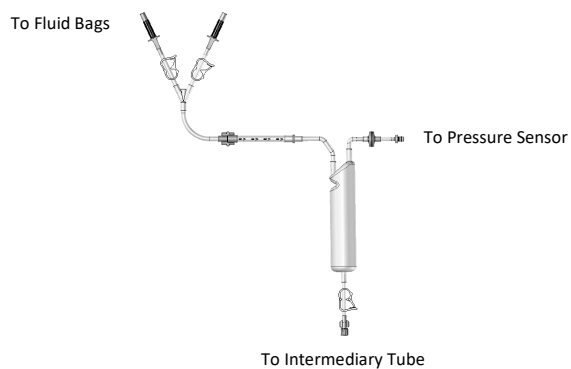

(EN)	DAY SET for AQUA VISION®	3
(ES)	SET DIARO para AQUA VISION®	4
(IT)	SET GIORNALIERO per AQUA VISION®	5
(DE)	TAGES-SET für AQUA VISION®	6
(FR)	SET JOURNALIER pour AQUA VISION®	7
(NL)	DAGELIJKS SET voor AQUA VISION®	8
(PT)	TUBO DIÁRIO para AQUA VISION®	9
(DA)	DAGSÆT til AQUA VISION®	10
(NO)	DAGSETT for AQUA VISION®	11
(FI)	PÄIVÄSARJA AQUA VISION®	12
(EL)	ΣΕΤ ΚΑΘΗΜΕΡΙΝΑ για AQUA VISION®	13
(SV)	DAGSUPPSÄTTNING till AQUA VISION®	14
(PL)	ZESTAW DZIENNY DO AQUA VISION®	15

DAY SET for AQUA VISION®
REF. AV4503

ENGLISH

INSTRUCTIONS FOR USE

INDICATIONS FOR USE: The device for daily use delivers the irrigation fluid from the bag to the intermediary irrigation tube. This device is used in conjunction with the intermediary irrigation tube or patient set (AV1102 or AV4509) and the Hemodia arthroscopic AQUA VISION® pump.

DESCRIPTION: Daily irrigation tube DAY SET for AQUA VISION® pump.

SUPPLY DETAILS: Sterile, if the package is unopened and undamaged. Re-usable for the surgical day when used with Ref. AV1102 or AV4509. See warnings below and appropriate AQUA VISION® pump user manual.

INSTRUCTIONS FOR USE:

1. Open using aseptic technique.
2. Open the outer package.
3. Close all three clamps and connect the spikes to the irrigation fluid bags.
4. The circulating nurse slips the green connector from the tubing into the green notch.
5. Place and center the tubing around the roller head.
6. Pull the tube tight.
7. Place the pressure chamber (vertically) in holder, close the pressure arm against the tube and place the transducer of the tubing onto the CPC™ connector of the pump.
8. Open one of the clamps connected to the irrigation fluid bags.
9. Close the left cover ("INFLOW") on the pump.
10. Press on the "Run/Stop" button of the AQUA VISION® pump. Blinking green light will turn to solid green light.
11. Press the "Fill Chamber" button, several times, until chamber reaches 1/3 full.

WARNING: The pressure chamber must remain vertical during the procedure.

WARNING: The DAY SET tube can be re-used for multiple procedures during the same surgical day only when used in conjunction with Ref. AV1102 or Ref. AV4509.

WARNING: Sterility is preserved only if proper setup and tube changing procedures have been followed according to the AQUA VISION® pump instruction manual.

WARNING:

1. Do not use if the package is damaged or opened.
2. Refer to and read the AQUA VISION® pump instruction manual for proper use.
3. Re-sterilization:
This product has not been designed to be re-sterilized. Reprocessing may lead to changes in material characteristics such as deformation and material degradation which may impact the strength of the device and compromise device performance. These risks may potentially affect patient safety.

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