INSTRUCTIONS FOR USE

SINGLE SUCTION TUBE

for AQUA VISION®

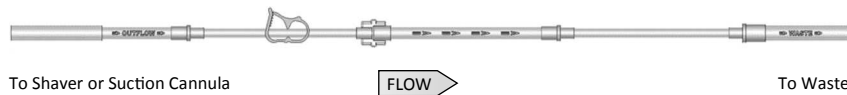
REF. AV4505

Hemodia SAS
85 Rue du Chêne Vert
31670 Labège - FRANCE
hemodia@hemodia.com

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(EN)	SINGLE SUCTION TUBE for AQUA VISION®	3
(ES)	ENTUBADO DE SUCCIÓN SENCILLO para AQUA VISION®	4
(IT)	TUBO DI ASPIRAZIONE SINGOLO per AQUA VISION®	5
(DE)	EINMAL-ABSAUGSCHLAUCH für AQUA VISION®	6
(FR)	TUBE D'ASPIRATION SIMPLE pour AQUA VISION®	7
(NL)	EENMALIGE AANZUGSLANG voor AQUA VISION®	8
(PT)	TUBO DE ASPIRAÇÃO ÚNICO para AQUA VISION®	9
(DA)	ENKELTSUGESLANGE til AQUA VISION®	10
(NO)	ENKEL SUGESLANGE for AQUA VISION®	11
(FI)	YKSITTÄINEN IMUPUTKI AQUA VISION® varten	12
(EL)	MONO ΣΩΛΗΝΑ ΑΝΑΡΡΟΦΗΣΗΣ για AQUA VISION®	13
(SV)	ENKELSUGSLANG till AQUA VISION®	14

SINGLE SUCTION TUBE
for AQUA VISION®
REF. AV4505

ENGLISH

INSTRUCTIONS FOR USE

INDICATIONS FOR USE: The device is intended to be used in conjunction with the Hemodia arthroscopic AQUA VISION® pump during arthroscopic procedure. The suction tubing is used to remove the fluid in the joint via the cannula or a shaver.

DESCRIPTION: SINGLE SUCTION TUBE for AQUA VISION® pump.

SUPPLY DETAILS: Sterile, single use. Sterile if the package is unopened and undamaged.

INSTRUCTIONS FOR USE:

1. Open using aseptic technique.
2. Open the outer package.
3. The scrub nurse removes the tube set from the package.
4. The scrub nurse takes the SINGLE SUCTION TUBE and hands the tubing end marked "WASTE" to the circulating nurse.
5. Insert the "WASTE" end into the waste collection device.
6. The circulating nurse slips the orange connector of the tubing into the orange notch. The label, stuck on the orange connector, must be located against the pump.
7. Place and center the tubing around the roller head.
8. Pull the tube tight, place it into the housing provided for this purpose and close the pressure arm against the tube.
9. Insert the tubing into the right side of the pinch valve if using a cannula.
Press the "Pinch Valve" button of the AQUA VISION® pump and then insert the tubing into the left side of the pinch valve if using a shaver.
10. Pass the tubing, after the pinch valve, into the hook provided for this purpose.
11. Close the right cover (Outflow) on the pump.
12. Plug the tubing end marked "OUTFLOW" onto the cannula or onto the shaver device.

WARNING:

1. Do not use if package is damaged or opened.
2. Refer to and read the AQUA VISION® pump instruction manual for proper use.
3. This product is for single-use only. It has not been designed to be re-used/re-sterilized. Reprocessing may lead to changes in material characteristics such as deformation and material degradation which may impact the strength of the device and compromise device performance. Reprocessing of single use devices can also cause cross-contamination leading to patient infection. These risks may potentially affect patient safety.

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