INSTRUCTIONS FOR USE

PATIENT SET
WITH CHECK VALVE
for AQUA VISION®
REF. AV4509


Hemodia SAS
85 Rue du Chêne Vert
31670 Labège - FRANCE
hemodia@hemodia.com

103789_D
Date de révision : 28 SEP 2023

PATIENT SET WITH CHECK VALVE
for AQUA VISION®








REF. AV4509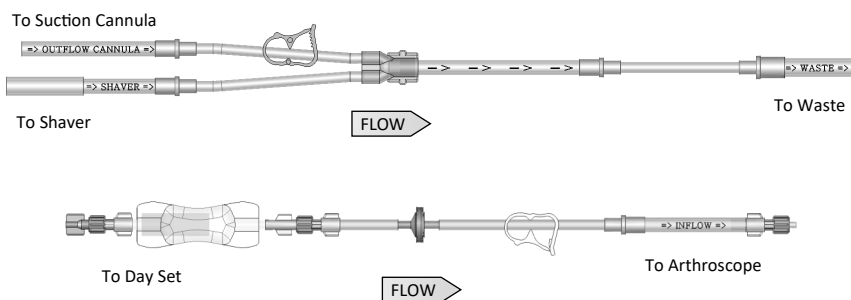

EN	PATIENT SET WITH CHECK VALVE for AQUA VISION®	4
ES	CONJUNTO PACIENTE CON VÁLVULA ANTIRRETORNO para AQUA VISION®	5
IT	SET PAZIENTE CON VALVOLA UNIDIREZIONALE per AQUA VISION®	6
DE	PATIENTEN-SET MIT RÜCKSCHLAGVENTIL für AQUA VISION®	7
FR	SET PATIENT AVEC VALVE ANTI-RETOUR pour AQUA VISION®	8
NL	PATIËNT SET MET EEN-WEGKLEP voor AQUA VISION®	9
PT	CONJUNTO PACIENTE COM VÁLVULA UNIDIRECCIONAL para AQUA VISION®	10
DA	PATIENT SÆT MED ÉNVEJSVENTIL til AQUA VISION®	11
NO	PASIENT SETT MED TILBAKESLAGSVENTIL for AQUA VISION®	12
FI	POTILASSARJA SULKUVENTTIILIN KANSSA AQUA VISION®	13
EL	ΣΕΤ ΑΣΘΕΝΗ ΜΕ ΒΑΛΒΙΔΑ ΑΝΤΕΠΙΣΤΡΟΦΗΣ για AQUA VISION®	14
SV	PATIENTSET MED BACKVENTIL till AQUA VISION®	15
PL	ZESTAW PACJENTA Z ZAWOREM ZWROTNYM DO POMPY AQUA VISION®	16

PATIENT SET WITH CHECK VALVE
for AQUA VISION®

REF. AV4509

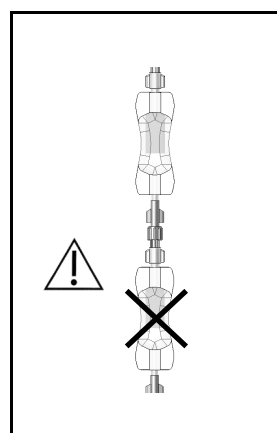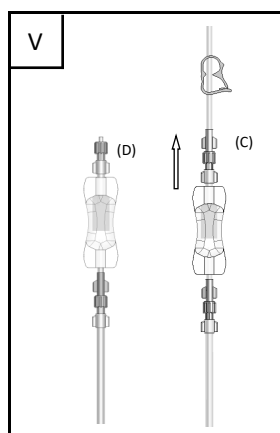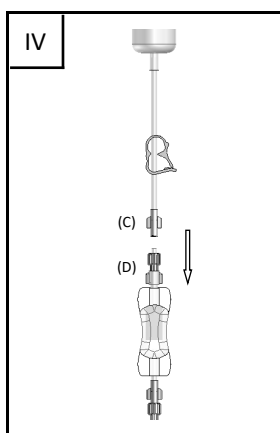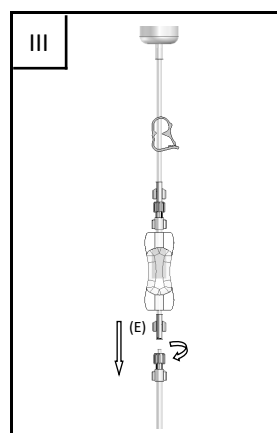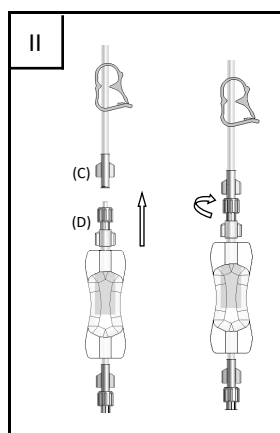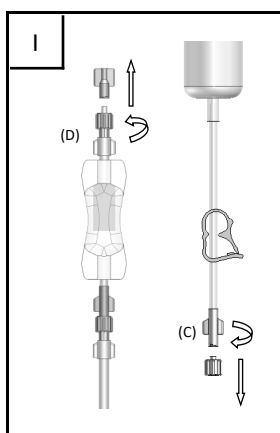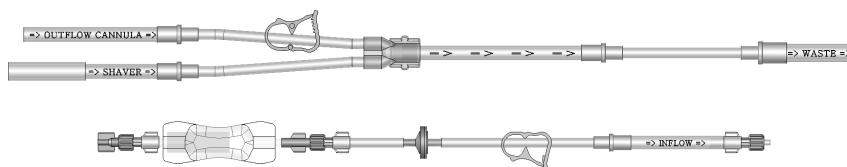PATIENT SET WITH CHECK VALVE
for AQUA VISION®
REF. AV4509

ENGLISH

INSTRUCTIONS FOR USE

INDICATIONS FOR USE: The device is intended to be used in conjunction with the Hemodia arthroscopic AQUA VISION® pump during arthroscopic procedure and is composed of an intermediary tube (AV1102) and a suction tubing. The intermediary tube delivers the irrigation fluid from the DAY SET tube (AV4503) to the arthroscope. The suction tubing is used to remove the fluid in the joint via the cannula or a shaver.

DESCRIPTION: PATIENT SET made up of intermediary & suction tube sets for AQUA VISION® pump.

SUPPLY DETAILS: Sterile, single use. Sterile if the package is unopened and undamaged.

INSTRUCTIONS FOR USE:

1. Open using aseptic technique.
2. Open the outer package.
3. The scrub nurse removes the both tube sets from the package.
4. Remove the smaller tube set (INTERMEDIARY) from the small package.
5. Ensure that all Luer locks are tight.
6. The scrub nurse hands the tubing end with the blue cap to the circulating nurse.
7. Remove the blue cap from the end of the INTERMEDIARY tubing (Fig I. D) and the DAY SET tubing (Fig I. C).
8. Immediately connect the INTERMEDIARY tubing (Fig II. D) and the DAY SET tubing (Fig II. C). Tighten the nut to lock the both Luer connectors.
9. Press on the "Run/Stop" button of the AQUA VISION® pump. Blinking green light will turn to solid green light. Open the clamp under the fill chamber to prime tubing.
10. Once the tubing has been primed, close off clamp.
11. The scrub nurse takes the large tube set (suction tubing) and hands the tubing end marked "WASTE" to the circulating nurse.
12. Insert the "WASTE" end into the waste collection device.
13. The circulating nurse slips the orange connector of the tubing into the orange notch.
14. Place and center the tubing around the pump head.
15. Pull the tube tight, place it into the housing provided for this purpose and close the pressure arm against the tube.
16. Insert the right tubing "OUTFLOW CANNULA" into the right side of the pinch valve. Press the "Pinch Valve" button of the AQUA VISION® pump, then insert the tubing marked "SHAVER" into the left side of the pinch valve.
17. Pass the tubes, after the pinch valve, into the hook provided for this purpose.
18. Close the right cover ("OUTFLOW") on the pump.
19. The scrub nurse plugs the tubing end marked "OUTFLOW CANNULA" onto the cannula. Plug the tubing end marked "SHAVER" onto the shaver device.

END of the operation:

- Close the clamp under the pressure chamber.
- Disconnect the white Luer lock (Fig III. E) under the check valve and discard tubing. The check valve will remain connected to the DAY SET tubing until the start of the next operation.

START of the next operation:

1. The circulating nurse removes the old check valve by disconnecting the used check valve (FIG IV. D) from the DAY SET tubing (FIG IV. C).
2. Immediately connect the blue Luer lock (Fig. V. D) of the new INTERMEDIARY tubing to the blue connector (FIG V. C) of the DAY SET tubing as per step 7 in the Instructions for use.

WARNING:

1. Do not use if the package is damaged or opened.
2. Refer to and read the AQUA VISION® pump instruction manual for proper use.
3. This product is for single-use only. It has not been designed to be re-used/re-sterilized. Reprocessing may lead to changes in material characteristics such as deformation and material degradation which may impact the strength of the device and compromise device performance. Reprocessing of single use devices can also cause cross-contamination leading to patient infection. These risks may potentially affect patient safety.
4. Never attach one check valve to another.

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