INSTRUCTIONS FOR USE

DAY SET

FOR DUAL FLOW PUMP

REF. 11010076

Distributed by
 ZIMMER BIOMET

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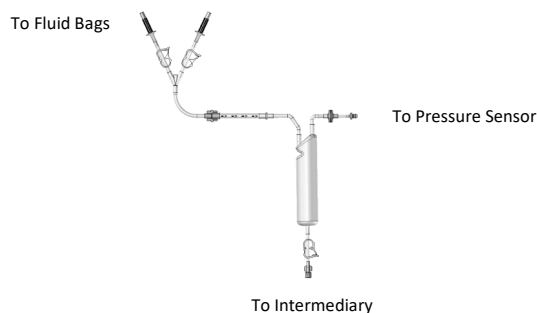
ENGLISH

INSTRUCTIONS FOR USE

INDICATIONS FOR USE: The device for daily use delivers the irrigation fluid from the bag to the intermediary irrigation tube. This device is used in conjunction with the Intermediary irrigation tube or Patient Set (Ref. 11010078 or Ref. 11010077) and the Dual Flow Pump (Ref. 11010072).

DESCRIPTION: Daily irrigation tube DAY SET for Dual Flow Pump.

SUPPLY DETAILS: Sterile, if the package is unopened and undamaged. Re-usable for the surgical day when used with Ref. 11010078 or Ref. 11010077. See warnings below and appropriate Dual Flow Pump user manual (Document 104072).

INSTRUCTIONS FOR USE:

1. Open using aseptic technique.
2. Open the outer package.
3. Close all three clamps and connect the spikes to the irrigation fluid bags.
4. Slip the green connector from the tubing into the green bracket.
5. Place and center the tubing around the roller head.
6. Pull the tube tight.
7. Place the pressure chamber (vertically) in holder, close the pressure arm against the tube and place the transducer of the tubing onto the CPC™ connector of the pump.
8. Open one of the clamps connected to the irrigation fluid bags.
9. Close the left cover ("INFLOW") on the pump.
10. Press on the "Run/Stop" button of the Dual Flow Pump. Blinking green light will turn to solid green light.
11. Press the "Fill Chamber" button, several times, until chamber reaches 1/3 full.

WARNING: The pressure chamber must remain vertical during the procedure.

WARNING: The DAY SET tube can be re-used for multiple procedures during the same surgical day only when used in conjunction with Ref. 11010078 or Ref. 11010077.

WARNING: Sterility is preserved only if proper setup and tube changing procedures have been followed according to the Dual Flow Pump instruction manual.

WARNING:

1. Do not use if the package is damaged or opened.
2. Refer to and read the Dual Flow Pump instruction manual (Document 104072) for proper use.
3. Re-sterilization:
This product has not been designed to be re-sterilized. Reprocessing may lead to changes in material characteristics such as deformation and material degradation which may impact the strength of the device and compromise device performance. These risks may potentially affect patient safety.

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