INSTRUCTIONS FOR USE

PATIENT SET

WITH CHECK VALVE

FOR DUAL FLOW PUMP

REF. 11010077

Distributed by
 ZIMMER BIOMET

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
 0459

104111_D
Review date : 04/2024

PATIENT SET WITH CHECK VALVE
FOR DUAL FLOW PUMP








REF. 11010077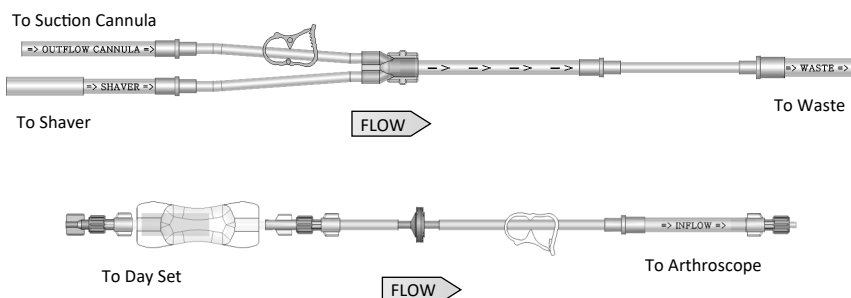

EN	PATIENT SET WITH CHECK VALVE FOR DUAL FLOW PUMP	4
ES	SET TUBOS PACIENTE CON VÁLVULA ANTIRRETORNO PARA BOMBA DE DOBLE FLUJO	5
IT	SET PAZIENTE CON VALVOLA UNIDIREZIONALE PER POMPA A DOPPIO FLUSSO	6
DE	PATIENTENSCHLAUCH-SET MIT PRÜFVENTIL FÜR DOPPELROLLENPUMPE	7
FR	TUBULURE SET PATIENT AVEC VALVE ANTI-RETOUR POUR POMPE À DOUBLE FLUX	8
NL	PATIËNTEN SET MET TERUGSLAGKLEP VOOR DUBBELE SCOPIE POMP	9
PT	KIT PACIENTE COM VALVULA UNIDIRECTIONAL PARA BOMBA DE DUPLO FLUXO	10
PL	ZESTAW PACJENTA Z ZAWOREM ZWROTNYM DO POMPY SSAĆO-PŁUCZĄCEJ	11
RO	SET PACIENT CU SUPAPĂ DE CONTROL PENTRU POMPA CU DEBIT DUBLU	12
EL	ΣΕΤ ΑΣΘΕΝΟΥΣ ΜΕ ΒΑΛΒΙΔΑ ΕΛΕΓΧΟΥ ΓΙΑ ΑΝΤΛΙΑ ΔΙΠΛΗΣ ΡΟΗΣ	13
CS	PACIENTSKÁ SOUPRAVA SE ZPĚTNÝM VENTILEM PRO PUMPU DUAL FLOW	14
TR	DUAL FLOW PUMP İÇİN KONTROL VALFLİ HASTA SETİ	15
RU	НАБОР ПАЦИЕНТА С РЕГУЛИРУЮЩИМ КЛАПАНОМ ДЛЯ ДВУХПОТОЧНОЙ ПОМПЫ	16
SK	SÚPRAVA PRE PACIENTA SO SPÄTNÝM VENTILOM PRE DUÁLNE PRIETOKOVÉ ČERPADLO	17
SL	BOLNIKOV KOMPLET Z NEPOVRATNIM VENTILOM ZA ČRPALKO Z DVOJNIM PRETOKOM	18
LT	PACIENTO RINKINYS SU ATGALINIŲ VOŽTUVŲ DVIGUBO SRAUTO SIURBLIUI	19

PATIENT SET WITH CHECK VALVE
FOR DUAL FLOW PUMP

REF. 11010077

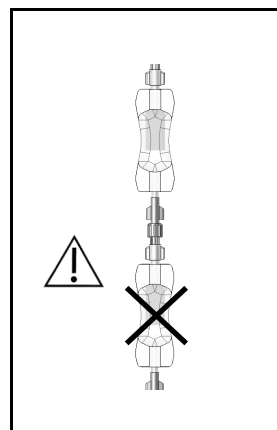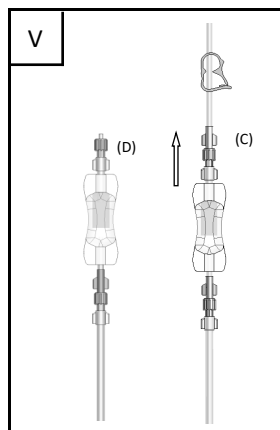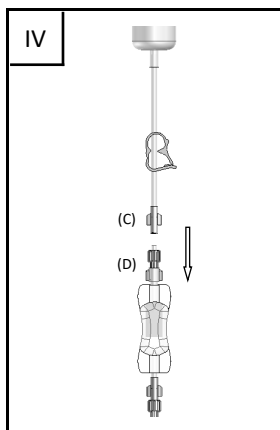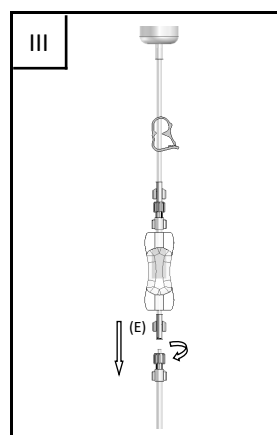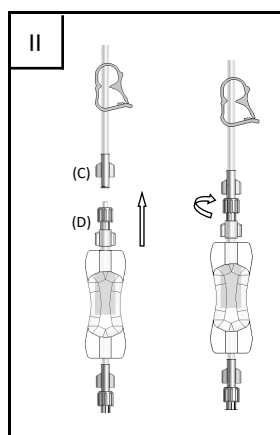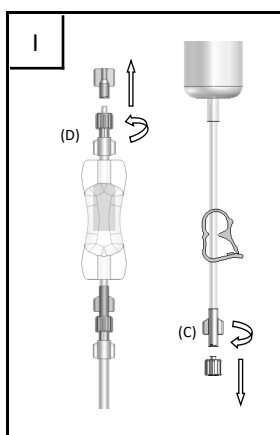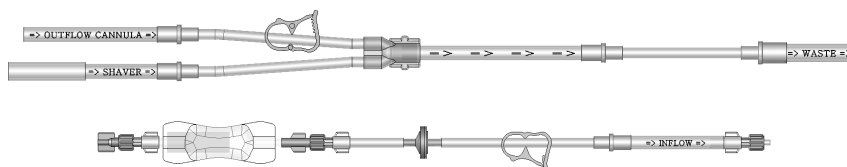PATIENT SET WITH CHECK VALVE
FOR DUAL FLOW PUMP
REF. 11010077

ENGLISH

INSTRUCTIONS FOR USE

INDICATIONS FOR USE: The device is intended to be used in conjunction with the Dual Flow Pump (Ref. 11010072) during arthroscopic procedure and is composed of an intermediary tube (Ref. 11010078) and a suction tubing. The intermediary tube delivers the irrigation fluid from the DAY SET tube (Ref. 11010076) to the arthroscope. The suction tubing is used to remove the fluid in the joint via the cannula or a shaver.

DESCRIPTION: PATIENT SET made up of intermediary & suction tube sets for Dual Flow Pump.

SUPPLY DETAILS: Sterile, single use. Sterile if the package is unopened and undamaged.

INSTRUCTIONS FOR USE:

1. Open using aseptic technique.
2. Open the outer package.
3. The scrub nurse removes the both tube sets from the package.
4. Remove the smaller tube set (INTERMEDIARY) from the small package.
5. Ensure that all Luer locks are tight.
6. The scrub nurse hands the tubing end with the blue cap to the circulating nurse.
7. Remove the blue cap from the end of the INTERMEDIARY tubing (Fig I. D) and the DAY SET tubing (Fig I. C).
8. Immediately connect the INTERMEDIARY tubing (Fig II. D) and the DAY SET tubing (Fig II. C). Tighten the nut to lock the both Luer connectors.
9. Press on the "Run/Stop" button of the Dual Flow Pump. Blinking green light will turn to solid green light.
- Open the clamp under the fill chamber to prime tubing.
10. Once the tubing has been primed, close off clamp.
11. The scrub nurse takes the large tube set (suction tubing) and hands the tubing end marked "WASTE" to the circulating nurse.
12. Insert the "WASTE" end into the waste collection device.
13. The circulating nurse slips the orange connector of the tubing into the orange notch.
14. Place and center the tubing around the pump head.
15. Pull the tube tight, place it into the housing provided for this purpose and close the pressure arm against the tube.
16. Insert the right tubing "OUTFLOW CANNULA" into the right side of the pinch valve. Press the "Pinch Valve" button of the Dual Flow Pump, then insert the tubing marked "SHAVER" into the left side of the pinch valve.
17. Pass the tubes, after the pinch valve, into the hook provided for this purpose.
18. Close the right cover ("OUTFLOW") on the pump.
19. The scrub nurse plugs the tubing end marked "OUTFLOW CANNULA" onto the cannula. Plug the tubing end marked "SHAVER" onto the shaver device.

END of the operation:

- Close the clamp under the pressure chamber.
- Disconnect the white Luer lock (Fig III. E) under the check valve and discard tubing. The check valve will remain connected to the DAY SET tubing until the start of the next operation.

START of the next operation:

1. The circulating nurse removes the old check valve by disconnecting the used check valve (FIG IV. D) from the DAY SET tubing (FIG IV. C).
2. Immediately connect the blue Luer lock (Fig. V. D) of the new INTERMEDIARY tubing to the blue connector (FIG V. C) of the DAY SET tubing as per step 7 in the Instructions for use.

WARNING:

1. Do not use if the package is damaged or opened.
2. Refer to and read the Dual Flow Pump instruction manual (Document 104072) for proper use.
3. This product is for single-use only. It has not been designed to be re-used/re-sterilized. Reprocessing may lead to changes in material characteristics such as deformation and material degradation which may impact the strength of the device and compromise device performance. Reprocessing of single use devices can also cause cross-contamination leading to patient infection. These risks may potentially affect patient safety.
4. Never attach one check valve to another.

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