

STERILE 0.9 % SOLUTION OF SODIUM CHLORIDE, IN UNIDOSE CONTAINERS

Nasal, ocular and auricular solution / Solution for cleansing wounds

DIRECTIONS AND PRECAUTIONS FOR USE

HOW IS sterile 0.9 % solution of sodium chloride USED ?

Do not use the same single-dose for the different uses.

The single-dose container is disposable and for individual use.

Detach a vial and open it by twisting off the top.

For nasal and ocular hygiene :

Sterile 0.9 % solution of sodium chloride may be used daily as a nasal and eye wash 1 to 6 times a day, as needed.

- For nasal hygiene :

To irrigate or cleanse dry or stuffed up noses.

- Directions for use :

Tilt the head back. Delicately insert the tip into one nostril and gently squeeze the vial. Repeat the operation in the other nostril. Lift the head up to allow the mucous to run out and then wipe away the excess. Do not blow your nose violently for 10 minutes after administering.

- Precaution for use :

For babies, squeeze drops with a minimum amount of pressure to avoid any risk of contamination of the middle ear.

- For ocular hygiene :

As an eyewash or bath to cleanse the eyes.

- Directions for use :

Pour a few drops of solution into each eye, taking care not to touch the surface of the eye with the vial. Wipe away excess liquid with a gauze.

-Precaution for use :

Wait 15 minutes before using any other eye treatment product.

For ear hygiene :

By rinsing.

For perfect ear hygiene, use the product to flush out solubilised earwax deposits and rinse the external auditory canal after using a surfactant auricular solution.

Directions for use :

Rinse the external auditory canal with sterile 0.9 % solution of sodium chloride.

Use 1 vial of sterile 0.9 % solution of sodium chloride for both ears.

Wipe away the surplus liquid with a gauze pad.

Precaution for use :

Ear use of this medical device in case of perforated eardrums, history of ear surgery or chronic ear conditions is not recommended or should be the subject of prior advice from your doctor or your pharmacist.

For cleansing wounds :**Directions for use :**

Rinse the wound, using the entire contents of one vial and taking care not to touch the wound with the vial.

Precaution for use :

Wipe away the excess liquid from the unaffected area only.

GB_DM_FT_092

Inception date : April 2021

Last modification date : April 2021


 Laboratoires Gilbert
 928 Avenue du Général de Gaulle
 14200 Hérouville Saint-Clair – FR.


 E0459

STERILE	A
----------------	----------

