


Device swabs 2 % CHG Prevantics®


Please follow local healthcare policy for personal protective equipment, hand hygiene & waste disposal guidelines.




Tear open the easily accessible packaging and remove swab.

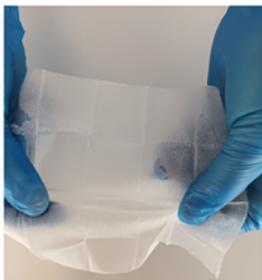

2- Unfold the wipe

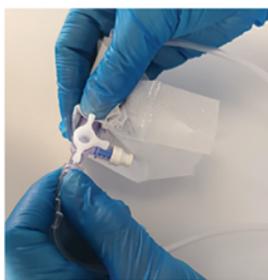

3. Twist back and forth in semi-circular movements for at least 30 seconds

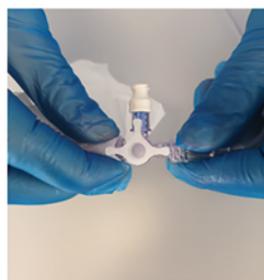

4. Ensure the connector is fully disinfected. Allow the needleless access site to air dry for 30 seconds



PDI Ltd. Pywel road,
Willowbrook East Ind Est,
Corby NN17 5XJ - UK



Nex Medical Antiseptics Srl.
Via Per Arluno 37/39
20010 Casorezo - IT



CE 2797

CAUTION :

Highly flammable liquid and vapour. Cause serious eye irritation. May cause drowsiness or dizziness. Very toxic to aquatic life with long lasting effects. Wear protective gloves.

IF IN EYES : Rinse cautiously for several minutes. Remove contact lenses; if present and easy to do. If eye irritation persists get medical advice. Keep product stored in a cool dry place away from heat source. Avoid release to the environment.

WARNING : Contains: Propan-2-ol

STORAGE :

Store between 20-25°C

Avoid freezing and excessive heat above 40°C

1004C 11-2021.