

Non woven adhesive strips

USAGE: This assembly is single-use and intended for a single patient

1- Prior to releasing the strips from the liner, destroy the backsplit on top and bottom of each sheet and on left and right sides.
2- Lift one edge of the product then slowly delaminate the tape in a straight move from the release liner. 3- Remove release liner remaining edges. 4- Gently press onto the tape to ensure proper adhesion to the skin. 5- At the end of dialysis session, remove slowly the tape from the skin.

CAUTIONS :

- Check the integrity of the packaging before use, do not use the device if the packaging has been damaged or opened before.
- Do not resterilize.
- Do not reuse: Reuse of the assembly may cause infection or harm to the patient.
- Check the expiry date on the product package or label. Do not use if the product has expired.
- Do not use for securing tubing used in haematology, emergency life support and incubators.
- Remove the tape slowly from the liner in a straight line, to avoid the “curling” phenomenon leading to a non-usable product for the medical staff.
- Do not use a curled tape or a tape where the adhesive part would have been exposed and/or touched.
- Remove any traces of topical product (disinfectant, local anesthetic, ointment, cream, etc.) and dry with a sterile gauze. Caution: residues of oily products (creams, ointments) may reduce adhesion on skin..
- Do not reuse or relocate a previously applied tape.
- Should be removed at the end of dialysis session.

STORAGE CONDITIONS : In original packaging, product will maintain stated properties when stored at ambient temperature, preferably in between 15°C and 30°C and in between 35% and 65% relative humidity .

MEDICAL DEVICE VIGILANCE:

Any serious incident occurring in connection with this device must be notified to the manufacturer or the relevant health authority.



ASSEMBLED BY :
Hemodia SAS
85 rue du Chêne vert
31670 LABEGE- FR

Creation date March 19, 2024