

Sterile Aquasonic® 100 Ultrasound Transmission Gel

Indications for Use

Sterile Aquasonic 100 Ultrasound Transmission Gel is a sterile, water-soluble and non-staining gel intended for sterile ultrasound procedures and where sterility is indicated.

Description

Sterile Aquasonic 100 Ultrasound Transmission Gel may be used:

- On or near broken skin, such as fresh surgical site or open wound where sterility is indicated.
- On mucous membranes and in natural body orifices.
- For patients where cross contamination is a concern such as:
- Patients with immunodeficiency or on immunosuppressive therapy.
- Neonates or critically ill pediatric patients.
- Critically ill patients or patients in contact, airborne or droplet transmission-based precautions.

Instructions for Use

Apply Sterile Aquasonic 100 Ultrasound Transmission Gel to transducer surface, not directly on the patient. To reduce the risk of contamination and infection, when performing sterile procedures, a sterile probe cover and sterile ultrasound gel should be used. Remove or flush gel from procedure site. Discard product after use.

Contraindications

None

Storage Conditions

Store in a cool dry place. Store away from direct sunlight.

Warnings and Precautions

Do not use on patients with known sensitivity to parabens.

Discard product if package is damaged or evidence of contamination is present.

To avoid contamination, recommended practices for sterile technique should be used.

Sterile Aquasonic 100 Ultrasound Transmission Gel is for single use only. Re-use of the device could result in cross-contamination and risks of infection or adverse reaction for the patient.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established

“Caution: Federal law restricts this device to sale by or on the order of a physician or a practitioner trained in its use.”

PM-010-0002F - 2023


Internal drafting date 06/08/2026



Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004 USA



Parker Laboratories BV
Bedrijvenpark Twente 305
7602 KL Almelo - NL

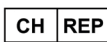

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau - Switzerland

