

KOCHER FORCEPS

Instruction for use AA78

Artery forceps are used in almost any surgical procedure to hold tissue during an operation or to compress bleeding tissue, control bleeding, and clamp ruptured blood vessels.

TESTING AND INSPECTION:

Inspect for rust, pits, cracks, pores, and dents before use.

WARNINGS:

Don't use for a second time.

Don't use if you observe sharp edges.

Product material is conductive : take away from electric source.

Do not use the wrong instrument, as it may cause problems during operation.

The use of the Kocher forceps for clamping the umbilical cord must be done under constant supervision to prevent any risk of unclamping and should be replaced as soon as possible by a Bahr clamp

STORAGE & TRANSPORT CONDITIONS :


Ambient temperature range : 0°C to 50°C (32°F to 122°F)- Relative humidity range : 30% to 75% - Atmospheric pressure range : 700 hPA to 1060 hPA

RECYCLE:

When processing medical devices please follow your local government Health & Safety procedures.

Doc. # IFU-02-05/01 dated 07/11/2023

Internal drafting date 06/08/2026

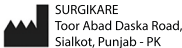

Obelis
Bd Général Wahis, 53
B-1030 Brussels - BE

