

USER GUIDE SAFE SCALPEL

Instruction for use AB01_EN

Usage tips:

- Handle the scalpel between your thumb and index finger. - At the same time, press the push button and slide the blade towards the end of the handle until it locks. - Use the scalpel without pressing the push button during the incision. - To temporarily retract the blade, slide the control into the body of the handle until it locks at the first notch of the slide. A click indicates that the blade is temporarily retracted. - To retract the blade permanently, carry out the same step by sliding the control all the way to the bottom of the handle until the blade is permanently locked. A click then indicates that the blade is blocked and secured. - Dispose of the single-use retractable scalpel in a waste collection box.

Support plan : A Swann-Morton delegate is available to assist you and answer all your technical questions, by in-person or remote appointment. A video tutorial is available at : <https://www.swann-morton.com>

Fiche technique n°19.3 version 2023 / 2024.



Swann Morton
Owlerton Green
Sheffield, S6 2BJ, England



Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem - NL



Internal drafting date 30/04/2024