



CAUTION: Where local and/or institutional procedures permit/require transportation of the filled syringe, use a passive recapping technique to cover the needle before transporting to the point of administration. Needle stick with a contaminated needle may lead to serious illness such as HIV, AIDS, Hepatitis and other infectious diseases. Do not force the shield onto the needle, as the needle may penetrate the shield causing injury. Seat the needle firmly on the syringe with a push and a clockwise twist. Pull the shield straight off. Re-use may lead to infection or other illness/injury. Do not resterilize. Pharmaceutical products prepared into these syringes should be administered as soon as possible.

• Verify individual package integrity. Do not use if individual packaging is damaged. Non pyrogenic. Not for Oral Use. Discard in approved sharps collector after single use.

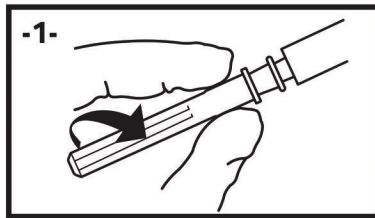

Push while turning

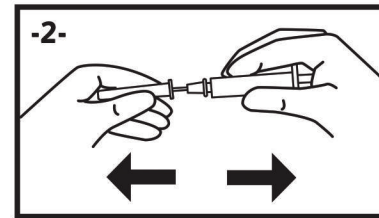

Pul horizontally

DGF401 Rev.07 du 31/01/2022