


Syringe. Where validated and claimed by a pump manufacturer, use a Luer Lock syringe, and refer to pump manufacturer instructions for operation. Ref. 300613 Not designed for use in syringe pumps. Verify individual package integrity. Do not use if individual packaging is damaged. Non pyrogenic. Not for Oral Use. Discard in approved sharps collector after single use.

Intended use: (Europe/CE Mark) BD Plastipak™ Syringe without Needle is used for general purpose aspiration and injection of fluids from vials, ampoules and parts of the body below the surface of the skin. • Accuracy: ISO 7886-1. **Intended User:** To be used by medical practitioners (e.g., physicians, nurses, pharmacists) experienced in the use of the device. Experience levels will be from novice to expert. **WARNINGS:** Do not use if the product or package has visible particulates. • Do not use if individual packaging is damaged, non-sterile product use may lead to infection. • Re-use may lead to infection or other illness/injury. • Where validated and claimed by a pump manufacturer, use a Luer Lock Syringe, and refer to pump manufacturer instructions for operation. • **Potential Procedure Related Complications:** Pain, exposure to bodily fluids and/or bloodborne pathogens, potential for infiltration/extravasation, particulate embolism, air embolism, localized/systemic infection, floaters, death. **PRECAUTIONS:** Prior to use verify information on package label. • Not using aseptic technique and failure to follow institutional guidelines may lead to infection. • To prevent leakage resulting in an inaccurate dose or exposure to medicine and bodily fluids; use only with ISO Compliant Luer Connectors and ensure secure connection. Do not use if product is damaged. • These syringes are not validated by BD for procedures with specific requirements such as, but not limited to, intraocular use and ear irrigation. Follow your institution's protocol when making device selection for specific applications. • Pharmaceutical products prepared into these syringes should be administered as soon as possible. • Dispose of device in accordance with national/institutional guidelines. **CAUTION:** Where local and/or institutional procedures permit/require transportation of the filled syringe, use a passive recapping technique to cover the needle before transporting to the point of administration. Needle stick with a contaminated needle may lead to serious illness such as HIV, AIDS, Hepatitis and other infectious diseases. Do not force the shield onto the needle, as the needle may penetrate the shield causing injury. Seat the needle firmly on the syringe with a push and a clockwise twist. Pull the shield straight off. Re-use may lead to infection or other illness/injury. Do not resterilize. Pharmaceutical products prepared into these syringes should be administered as soon as possible. EU Only: Users should report any serious incident related to the device to the Manufacturer and National Competent Authority.



Push while turning
DGF40109EV.2025-11

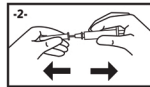

Pul horizontally