



Blunt Fill Needle 18 G x 1 1/2" (1.2 x 40 mm)

NOT FOR SKIN INJECTION

The BD Blunt Fill Needle is used for general purpose aspiration and injection of fluids from vials.

WARNINGS:

- Do not use if the product or package has visible particulates.
- Non-sterile product use may lead to infection. • Re-use may lead to infection or other illness/injury.
- REPORT NEEDLE STICKS IMMEDIATELY AND FOLLOW ESTABLISHED PROTOCOL. Percutaneous puncture with a contaminated needle may lead to serious illness such as hepatitis, HIV (AIDS), or other infectious diseases. Re-capping needles is hazardous.
- Potential Procedure Related Complications: Redness, irritation, pain, bleeding, anxiety, syncopal episode, granuloma, hematoma, paresthesia, exposure to bloodborne pathogens, potential for infiltration/extravasation, particulate embolism, localized/systemic infection, floaters.

PRECAUTIONS:

- Not for skin injection. • Prior to use verify information on package label. • Open the bag and process the product in a controlled area. • For manufacturing, processing, or repacking, Sterilize before use only with Ethylene oxide. • Please contact BD to obtain sterilization process information. • Not using aseptic technique and failure to follow institutional guidelines may lead to infection. • Use a filter needle when using glass ampoules. • Do not use if needle is bent.
- To prevent leakage resulting in an inaccurate dose or exposure to medicine and bodily fluids; use only with ISO Compliant Luer Connectors and ensure secure connection. Do not use if product is damaged. • Used needles may be contaminated and should be disposed of in accordance with national/institutional guidelines. • EU Only: Users should report any serious incident related to the device to the Manufacturer and National Competent Authority.

eifu.bd.com Rev. 2022-05

 Becton Dickinson S.A.,
Camino de Valdeoliva s/n
28750 San Agustín del Guadalix,
Madrid - ES

